

- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna



INSERT	POWER	IRRIGATION	MODE
<b>OSTEOTOMY</b>			
MT1-10	6 (3...7)	4 (3...6)	2
MT1S-10	4 (3...5)	4 (3...6)	3
MT1-20	6 (5...7)	4 (3...6)	3
MT2R-4	7 (3...7)	5 (4...6)	2
MT2L-4	7 (3...7)	5 (4...6)	2
MT6S-10	5 (4...6)	4 (3...6)	2
MT7-3	6 (3...7)	4 (3...6)	3
UNIVR	6 (5...7)	4 (3...6)	3
MT9-13	6 (4...7)	4 (3...6)	2
MT11R-10	7 (3...7)	5 (4...6)	2
MT11L-10	7 (3...7)	5 (4...6)	2
MT1204	6 (5...7)	4 (3...6)	3
<b>OSTEOPLASTY</b>			
MP1	6 (3...7)	4 (1...6)	3
MP2	6 (3...7)	4 (1...6)	2
MP3-a30	6 (3...7)	4 (1...6)	2
MP0745	6 (3...7)	4 (1...6)	3
<b>DRILLING</b>			
MD2-08	6 (5...7)	4 (3...6)	2
MD2-10	6 (5...7)	4 (3...6)	2
MD3-14	6 (4...7)	4 (3...6)	3
MD3-18	6 (4...7)	4 (3...6)	3
<b>FINISHING</b>			
MF1	6 (6...7)	4 (3...6)	2
MF2	6 (6...7)	4 (3...6)	2
MF3	6 (4...7)	4 (3...6)	2
MF4	5 (4...6)	4 (3...6)	2
MF5	7 (6...7)	4 (3...6)	2
MF6	2 (1...3)	4 (2...6)	1